

www.epcmfg.com

## **HOPE FACE SHIELD**

**Part Number 1672G-11** 

In response to the COVID-19 outbreak in 2020, EPC worked with the University of Iowa staff and has responded by making a mold and mass producing face shields to protect you and help limit the spread of the virus.



## EPC CUSTOM INJECTION MOLDER



## Assemble the Hope Face Shield kit by:

- Remove backing from tape
- Align clear shield to the headband starting from the center mark
- Firmly press top edge of shield to the headband
- Securely hook the rubber band to the back of the headband
- If tighter fit is desired, tie knots in the rubber band
- For looser fit, try looping two rubber bands together or use a tie string